

www.pawbrothers.com




TM


# Paw Brothers® Professional Modular Unit Cages (PBP89001 & PBP89002)

**Included Items:**


- |                            |   |                                 |    |
|----------------------------|---|---------------------------------|----|
| (A) Front Wall.....        | 1 | (J) Allen Head Bolt.....        | 24 |
| (B) Side Walls.....        | 2 | (K) Hex Bolt.....               | 2  |
| (C) Rear Wall.....         | 1 | (L) Allen Wrench.....           | 1  |
| (D) Ceiling.....           | 1 | (M) Double Ended Bolts.....     | 4  |
| (E) Floor.....             | 1 | <b>Optional Accessories</b>     |    |
| (F) Cross Bar.....         | 1 | (N) Casters (PBP89012).....     | 4  |
| (G) Floor Grates.....      | 2 | (O) Stabilizers (PBP89013)..... | 8  |
| (H) ABS Plastic Trays..... | 2 |                                 |    |
| (I) Divider Panel.....     | 1 |                                 |    |

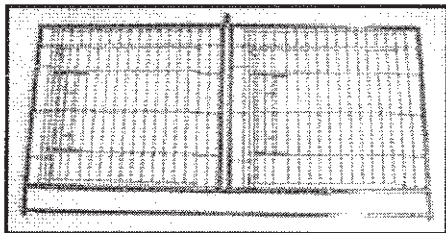


Front Wall (A)

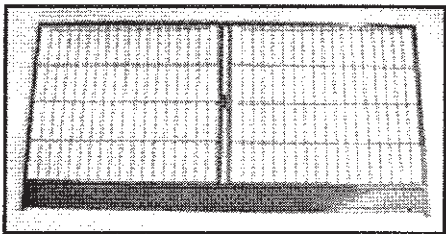


Rear Wall (C)

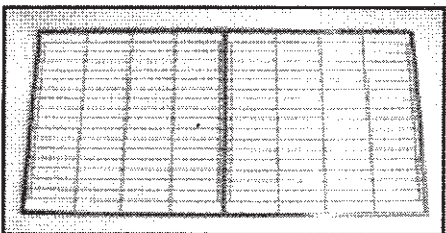


Ceiling (D)

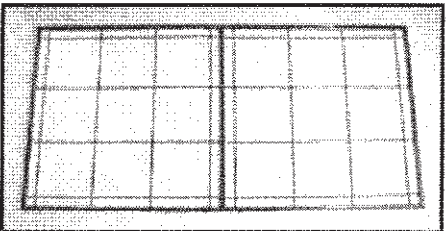


Floor (E)

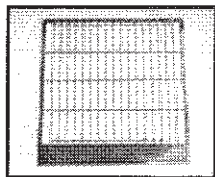


Side Wall (B)

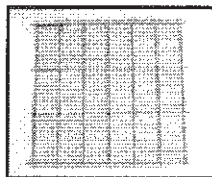


Floor Grate (G)

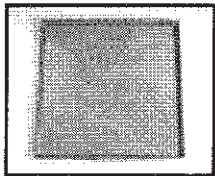


ABS Plastic Tray (H)

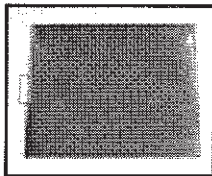

Divider Panel (I)

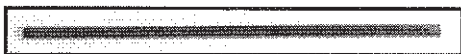

Cross Bar (F)

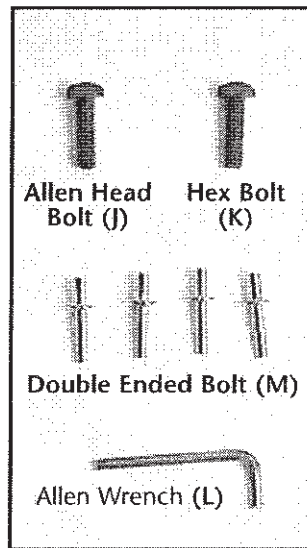


Allen Head Bolt (J)

Hex Bolt (K)


Double Ended Bolt (M)

Allen Wrench (L)

**Optional Accessories:**

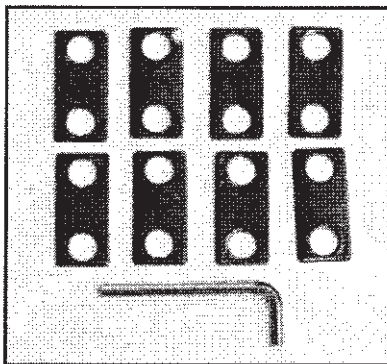

Stabilizers (O) - PBP89013

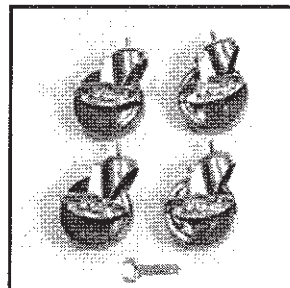


Casters (N)  
PBP89012



Includes "Thread Lock"

# Cage Assembly Instructions

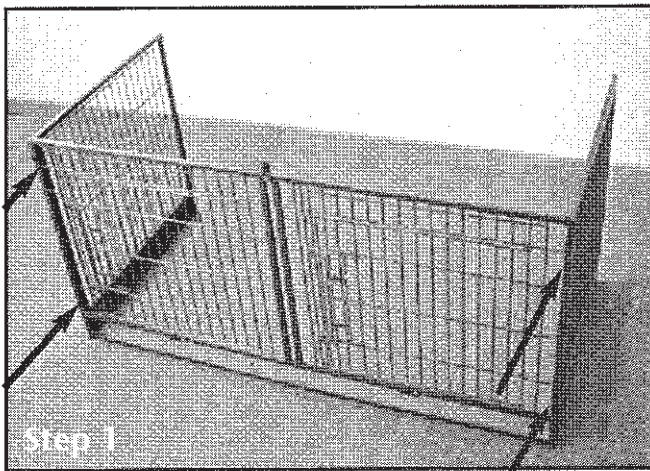

## Step 1:



With front wall (A) standing, attach side walls (B) with allen head bolts (J).

Note: Leave bolts loose till "Step 6".

## Step 2:

Attach rear wall (C) to side walls using 4 allen head bolts (J).

Note: Leave bolts loose till "Step 6".

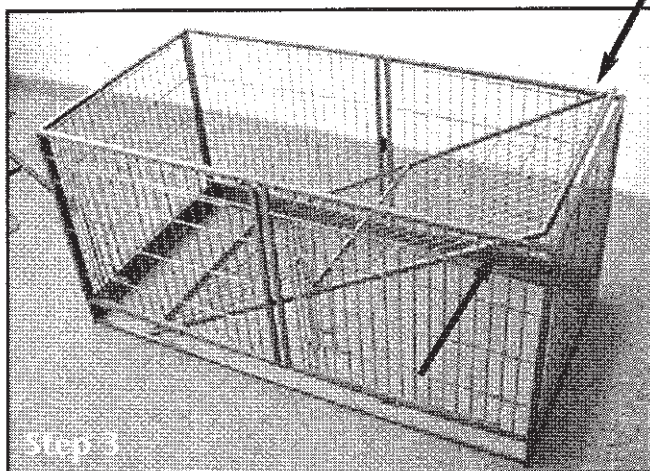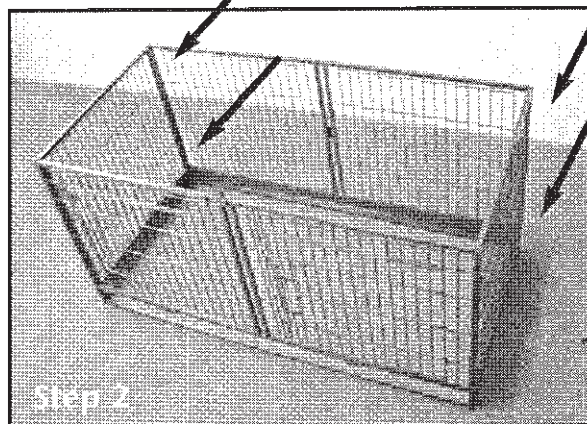


## Step 3:

Place the floor (D) into frame and attach with 2 allen head bolts (J) as shown.


Then raise the lowered end of the floor and attach it with 6 allen head bolts (J).

Note: Leave bolts loose till "Step 6".

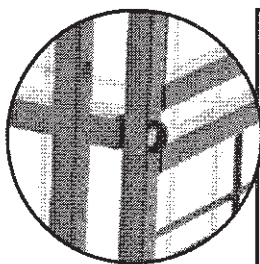

## Step 4:


Stand cage on it's end. Attach the cross bar (E) using 2 hex bolts (K).

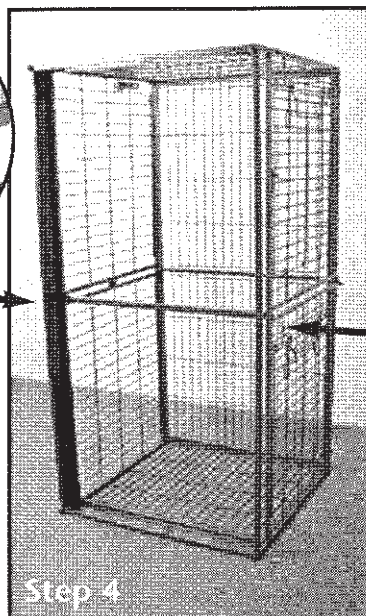


## Step 5:

Attach the floor (E) to the cage with 8 allen head bolts (J).

Note: Leave bolts loose till "Step 6".

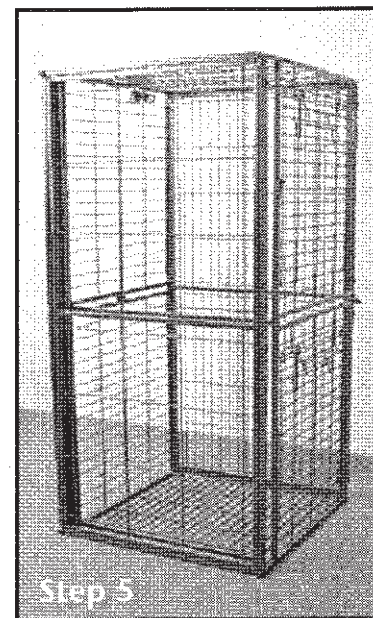



**Step 6:** Tighten all 24 allen head bolts in steps 1, 2, 3 and 5.

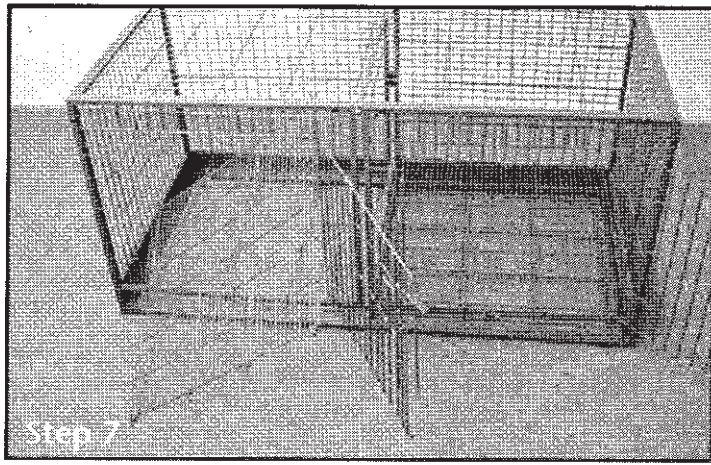

## Step 7:


Place cage on the floor. Then open the cage doors and place the 2 floor grates (G) into position.

Rotate latches open to allow trays to be inserted.

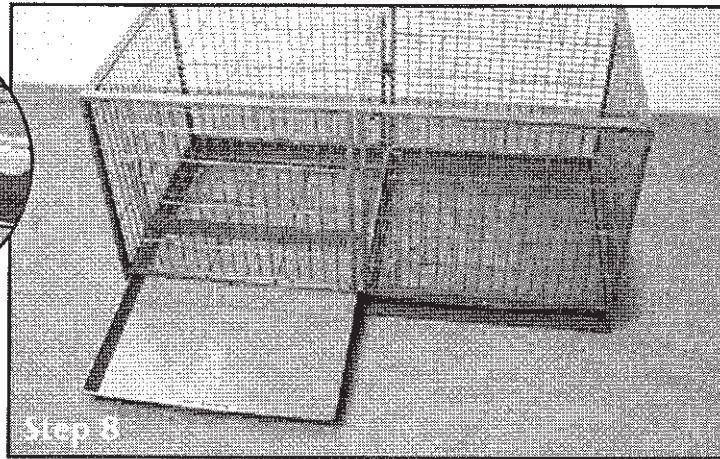
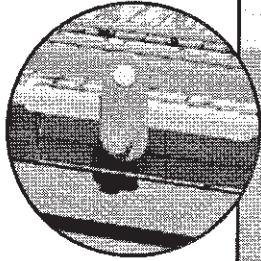


## Step 8:

Slide the 2 trays (H) into place.

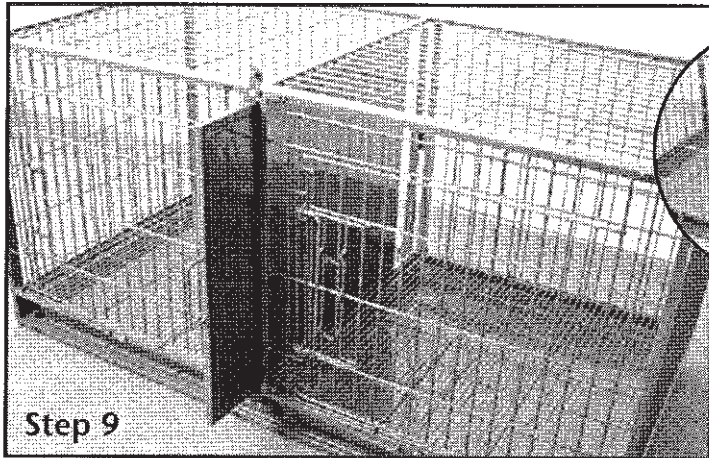


Rotate latches open to allow divider to be inserted.

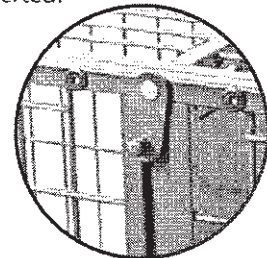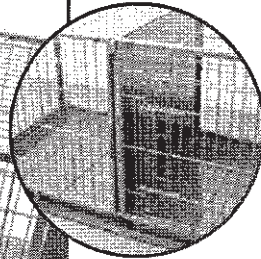


## Step 9:

Slide the divider panel (I) into place.

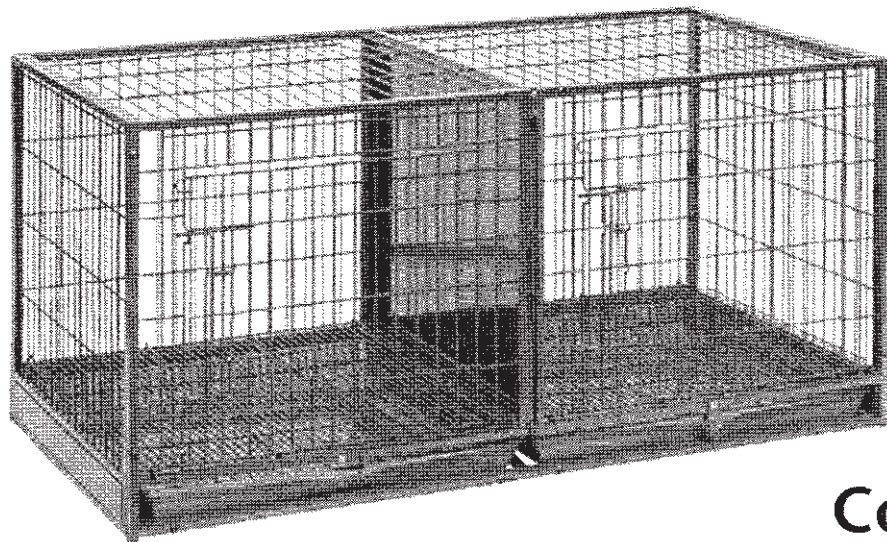


# Cage Complete!

# Optional Accessories

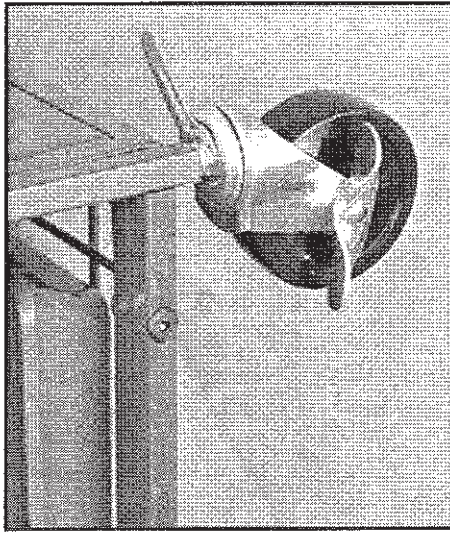

Note: Use "Thread Lock" gel to firmly secure the casters into place.



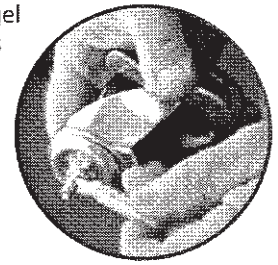


## Casters: (PBP89012)

- Step 1: Place cage on it's end.
- Step 2: Put provided "Thread Lock" gel on the screw portion of caster and hand screw it into the bottom corner hole.
- Step 3: Use provided wrench to tighten caster into position.
- Step 4: Repeat for remaining casters.

## Adding Second Tier:

Attach double-ended bolts (M) to each of the four upper corners of the lower tier. Lower second tier onto first tier so that the exposed halves of the bolts fit into the holes on the underside of the second tier.

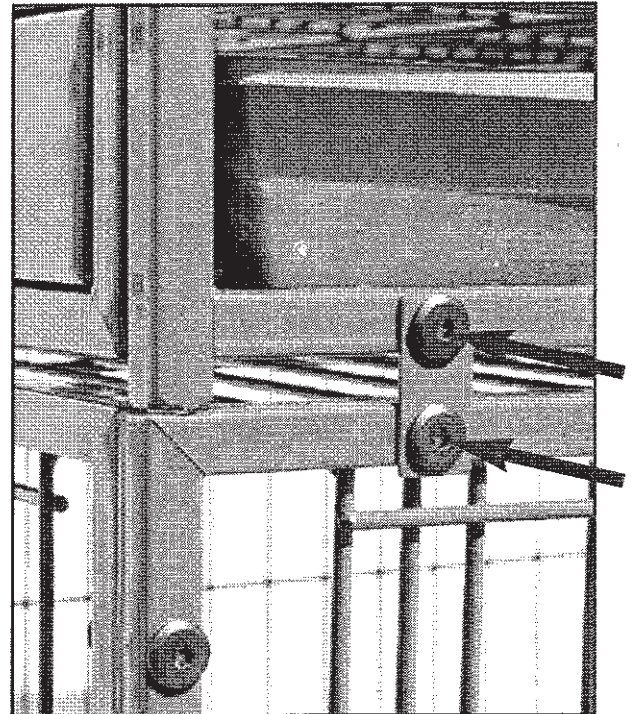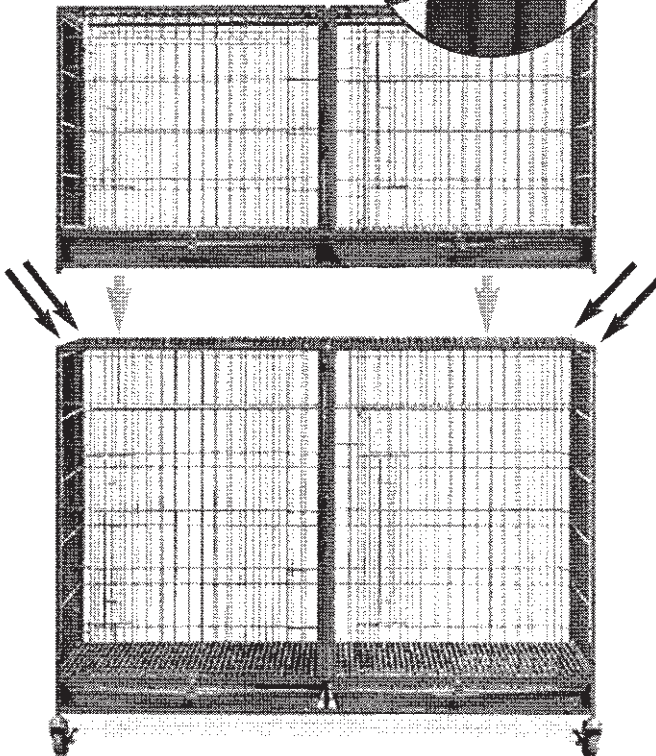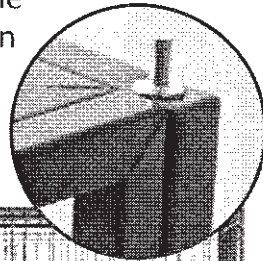

## Stabilizers: (PBP89013)

- Step 1: Remove two of the corresponding allen head bolts (J).
- Step 2: Place the stabilizer in position and replace the bolts into the cage loosely.
- Step 3: Repeat for the remaining stabilizer points.
- Step 4: Tighten all 16 allen head bolts.

## Paw Brothers® Professional Modular Unit Cages

For additional help with this or any Paw Brothers® product contact us at:  
1(800)525-7387 or Visit us at [www.PawBrothers.com](http://www.PawBrothers.com)